

Asepti BA Cleaner Disinfectant

Liquid disinfectant concentrate for disinfection of respiratory masks.



Asepti BA Cleaner Disinfectant is a disinfection and cleaning fluid Neutral pH, dual-chain quaternary ammonium-based one-step disinfectant, detergent, deodorant with broad spectrum bactericidal efficacy. 10 minute contact times for broad spectrum of microorganisms.

Advantages

- Blend of two quaternary Ammonium compounds
- Effective against bacteria, fungi, viruses, mould, mildew and prevents cross contamination
- Neutral PH
- Deodorizer

- Easy dosing with additional dosing unit
- High efficiency thanks to low usage concentrations in immersion baths

APPLICATION INSTRUCTIONS

- Use disinfectant with caution. Always read the label and product information before use.
- Strictly observe the respective Dräger instructions for use for respiratory masks, respiratory protection equipment, and the application instructions of the disinfectant manufacturer!



Asepti BA Cleaner Disinfectant
2 litre canister

DOSAGE

| | Concentration (ml/L) | Exposure time (min.) |
|--|-------------------------|-------------------------|
| Disinfection of respiratory protection devices and respiratory protection equipment in immersion baths | 12.5 | 10 |

DOSING TABLE – LIQUID

| Litre(s) | Concentration for use 1.25 % |
|-----------------|-------------------------------------|
| 1 litre | 12.5 ml |
| 2 litres | 25 ml |
| 3 litres | 37.5 ml |
| 4 litres | 50 ml |
| 5 litres | 62.5 ml |
| 8 litres | 100 ml |
| 10 litres | 125 ml |
| 20 litres | 250 ml |
| 30 litres | 375 ml |

Application instructions:

Example: 8 litres = 1.25 % application solution = 100 ml disinfectant + 7.9 litres water.

Add concentrated disinfectant according to the water volume, stir briefly. Solution can be used immediately.

Replace diluted solution every working day.

ORDER INFORMATION

| | | |
|--------------------------------|-------------------------------|---------|
| Asepti BA Cleaner Disinfectant | 2 litre container | 3505268 |
| Dosing unit | Manual pump adjustable dosing | 3504008 |
| Dosing pump | Manual pump (preset at 25mls) | 3504009 |

DISTRIBUTION

Draeger Safety Pacific Pty Ltd
8 Acacia Place
Notting Hill VIC 3168
Australia

Phone 1800 372 437
Fax 1800 010 327

www.draeger.com

DISTRIBUTION

Draeger Safety Pacific Pty Ltd
Unit 4, 24 Bishop Dunn Place
East Tamaki
Auckland 2013
New Zealand

Tel: 0800 372 437
Phone 1800 010 327